BENEfit®.KFO

www.psm.ms – premium implants

made in Germany

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**PSM Medical Solutions – The Premium Implant Manufacturer**

PSM Medical Solutions, established in 1993, was originally specializing in implants and instruments for the bony neurosurgical area. The company based in Tuttlingen, Germany, is part of a medical-economic cluster in and around Tuttlingen, the renowned world center of the medical industry. Many of the surgical instrument manufacturers have transferred their historic handmade craft into the high-tech companies known today.

The Tuttlingen region is known throughout Germany as the epicenter for medical scientific creativity. The local university offers specific courses in the study of the medical device industry. PSM has taken part in this evolution.

In addition to the PSM neurosurgical line, we are now pleased to offer a specialized implant, screw and instrumentation line for orthodontics/ oral surgery. Together with our international partners, PSM continues to grow as the trusted partner for clinicians world wide. PSM distinguishes itself from other manufacturers by combining practice, science, technology, material quality and excellent customer service in a cohesive way that truly advances the art and science of patient care.

PSM Medical Solutions – The Premium Implant Manufacturer

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premium
orthodontic
implants

INTRODUCTION EINFÜHRUNG

Skeletal anchorage in orthodontics

An important requirement for successful orthodontic treatment is stable anchorage. Conventional orthodontic anchorage can be limited by periodontal disease and/or missing teeth. Skeletal anchorage has been proven to be very effective. It can be an essential part of non-compliance therapy and broadens the options available to the orthodontic practitioner.

Mini-Implants are especially advantageous because of the minimally invasive insertion protocol and their relatively low cost.

However, limitations can occur when conventional mini-implant systems are used for orthodontic anchorage.

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PRODUCTS PRODUKTLÖSUNG

The **BENEFIT®**-System

The **BENEFIT®**-System can be used for a variety of applications in the maxilla – mainly in the palate – because of the design of the various abutments that are available. Indications include molar anchorage and distalization as well as anchorage for the retraction of anterior teeth and space closure.

BENEFIT® Mini-Implants can be placed in edentulous areas of the maxilla or mandible for aligning teeth prior to prosthodontics and they can be used for uprighting molars.

The abutment fixation screw is internally retained, as a safety measure to avoid aspiration by the patient.

The integration of the **BENEFIT®**-System in orthodontic practice will offer clinicians many new innovative treatment options.

Possible Indications:

- Maxillary molar distalization and mesialization
- Molar uprighting
- Anchorage for upper anterior teeth and molars
- Temporary tooth replacement
- Palatal expansion
- Indirect anchorage

Advantages:

- Significantly less expensive than typical dental implants
- Immediate loading is possible and recommended
- Minimally invasive insertion protocol
- Orthodontists can insert them themselves avoiding referrals to other dental specialists
- Easy to remove
- Internally retained fixation screw is securely integrated within each abutment

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ST
■ Note:
All implants are supplied in sterile single use packaging (see page 19).

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33-54512 20 mm **DENTAL**
33-54515 30 mm **DENTAL**
33-54518 50 mm **DENTAL**

Screwdriver insert for abutment retaining

screws **BENEplate** fixation screws and

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IMPRESSION TAKING
ABDRUCKNAHME

Packaging: 1ea.
VPE: 1Stk.

Note:
All products marked
with **QB** can be
used with **quattro®**
and **BENEFIT®** screws!

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The extension of 16 mm works for all rotary instruments of the
quattro® and **BENEFIT®**-System with a latch type DENTAL attachment
(max. rpm 600).

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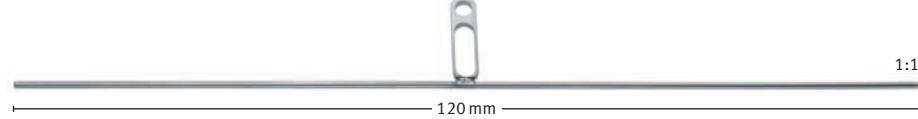
BENEplate, short, incl. fixation screws
BENEplate, kurz, inkl. Fixierschrauben

33-54429 1.1 stainless steel 1:1

BENEplate, long, with 1.1 mm wire (12 cm), incl. fixation screws

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33-54409 1.1 stainless steel 1:1

BENEplate, short, with 1.1 mm wire (12 cm), incl. fixation screws

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33-54428 0.8 stainless steel 1:1

BENEplate, long, with 0.8 mm wire (12 cm), incl. fixation screws

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33-54408 0.8 stainless steel 1:1

BENEplate, short, with 0.8 mm steel wire (12 cm), incl. fixation screws

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33-54407 1:1

BENEplate long, with bracket incl. fixation screws

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33-54420 0.8 TMA 1:1

BENEplate, short, with 0.8 mm TMA wire (12 cm), incl. fixation screws

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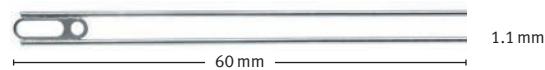
Note:
All **BENEplates** are supplied with 2 fixation screws

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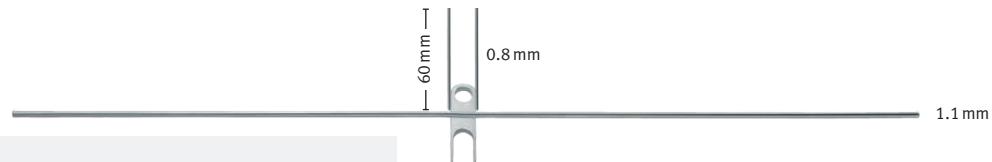
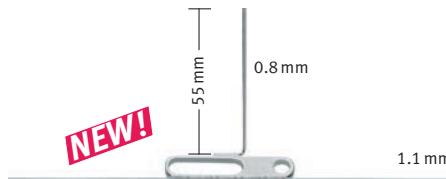
Packaging: 1 ea.
 VPE: 1 Stk.

Note:

All **BENEplates** are supplied with 2 fixation screws

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BENEFIT® Abutment with brackets

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33-54445

Abutment with loop acc. to Dr. S. Renger,
St. Louis, France

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33-54466

BENEFIT® Peek Abutment,
incl. 1 fixation screw

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Hyrax Ring, incl. fixation screws, 2 ea.

Hyrax Ring, inkl. Fixationsschrauben, 2Stk.

The new Hyrax Ring and Hyrax Abutment facilitate the manufacture and fixation of a Hybrid Hyrax appliance. It allows welding or soldering with the Hyrax Screw as well as little rotation movements when connecting the appliance to the BENEFIT Screws. The Hyrax Ring is supplied with 1 fixation screw each.

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33-54544

BENETube acc. to. Dr. Banach,
with wire 1.1 mm, 40 mm

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The BENEslder

The most classical indication for skeletal anchorage using Mini-Implants with abutments is molar-distalization and/ mesialization in the maxilla. By means of the so called „**BENEslder**“ the molars can be bodily distalized and/ or mesialized using the Mesialslider. In many cases, tooth extraction or dental implants can be avoided.

To couple two **BENEFIT®** Mini-Implants, a **BENEplate** with welded wire is connected to the Implants and the **BENEslder** parts (mobilizer, springs, **BENEtube**) are placed on the wire. Active force is applied by springs (240 g or 500 g) and activated with the Mobilizer.

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BENEtube, lang

33-54535

BENEtube Standard
for lingual brackets

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73-90000

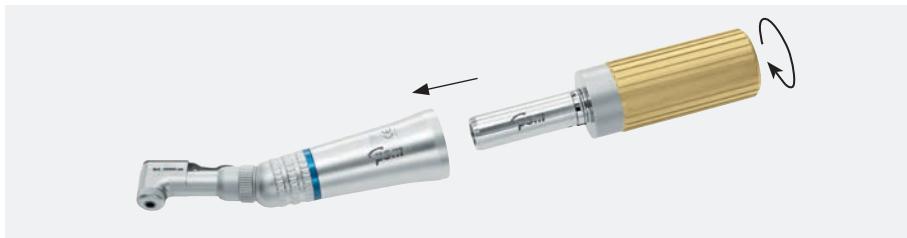
Utility storage box, without content,
not for sterilization, incl. forceps

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OPTIONAL INSTRUMENTS
OPTIONALES
INSTRUMENTARIUM

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use with an angled screwdriver.

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10-63025

Manually turned unit for contra-angled handpieces

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33-54704

Manually turned unit mod. to Pauls, with adjustable torque from 0 – 40 Ncm

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BENEslder NiTi springs, 2ea.
BENEslder NiTi Federn, 2Stk.

33-18288

Laboratory Finger Screwdriver, fixed
Labor Finger Schraubendreher, fest

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acc./nach Dr. Michael Schubert, Regensburg, Germany

33-54520

Multi purpose handle for the use with all instruments with dental shaft end.

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■ Note:

All products marked with **QB** can be used with **quattro**® and **BENEFit**® screws!

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■ Note:

*Set illustration can deviate from our set recommendation.

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95-13015

Model Hybridhyrax Distalizer 2
Modell Hybridhyrax Distalizer 2

BENEFIT® MODELS
BENEFIT® MODELLE
■ QR code:

■ Color:

Please select the desired **BENEFIT®** model/s and choose the color from above QR code file. If no color number is chosen we supply the model in Standard Color red (01).

■ Note:

Models can deviate in color and design from these illustrations.

The color must be selected individually for each model (See QR code).

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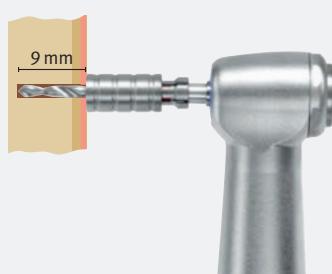
[33-18311](#)

[33-18313](#)
Stop sleeve
Stophülsen

The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate PSM drill*, the working length is adjusted to the desired depth. Removal of the sleeve is easy using the buffer.

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MOPS Bone-Rapper, 70 mm,
11 mm Gewinde

[ST-33-10133](#) **ST**

MOPS Bone-Rapper, 33 mm,
for contra angled handpiece

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ST
Note:
MOPS Bone-Rappers
are supplied in sterile
single use packaging.

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Cordless Battery Screwdriver with Torque Calibration System for the fixation of **quattro®** and **BENEFIT®** Miniscrews.

The iSD 900 will guarantee precise torque control due to the unique – easy to use – torque calibration system (10 – 40 Ncm). Time-saving and precise, risk-free insertion and tightening of screws and abutments.

Ask for the detailed brochure KAT-News 10.

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COURSES KURSE

psm MEDICAL SOLUTIONS offers courses in various language around the world.

Please check the currently available courses on our website:

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<http://www.psm.ms/veranstaltungen/>

quattro

The **quattro®** Mini Anchor System is known as one of the first and most successful orthodontic Mini-Implant Systems worldwide. Based on this system the **BENEFIT®** screws were developed. This enables the user to utilize the entire scope of orthodontic implantation with unmatched possibilities and compatible instrumentation.

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mentoplate

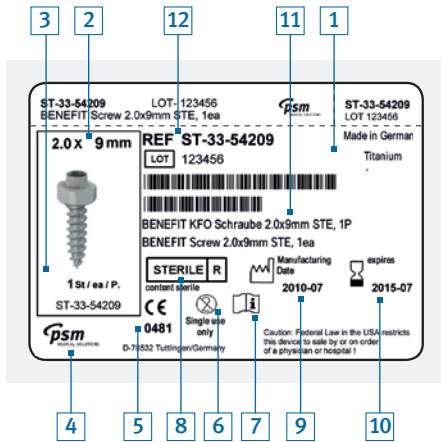
The Mentoplate-System provides a perfect solution for enhanced stability with special orthodontic bone anchored plates for intraoral fixation in the maxilla and the mandible. The System includes Titanium plates which are fixed to the bony surface with selfdrilling screws.

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Single sterile packaging
Sterile Einzelverpackung

1. Self-adhesive label for patient files
2. Product size (in mm)
3. Illustration and packaging
4. Manufacturer
5. Reference number of the notified body according to the directive 93/42 EEC
6. For single use only
7. Refer to instructions for use
8. Symbol for gamma sterilization
9. Date of manufacture
10. Sterility expiration date
11. Product description (german, english)
12. Reference number

**Implants
in single sterile packaging**

The outer and inner packaging contains labels indicating article number, LOT number, expiration date, etc. for the patient folder and for full traceability.

The sterile screws are contained in a small color-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can easily be removed from the carrier by means of the self retaining screwdriver.

Labels with product illustration for patient files

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Distribution | Vertrieb:

© 2016 psm MEDICAL SOLUTIONS | Germany

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of **psm** MEDICAL SOLUTIONS. Utilization and copies by third parties have to be authorized by **psm** MEDICAL SOLUTIONS. All rights reserved!

All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

Distribution in USA:

psm North America

P.O. Box 142
49950 Jefferson Street
Suite 130, Indio, CA 92201

Toll-free: 800-733-1622

service@psm-na.us
www.psm-na.us

Manufacturer | Hersteller:

psm MEDICAL SOLUTIONS

Moltkestraße 41
78532 Tuttlingen, Germany
Phone +49 (7461) 9 66 37-0
Fax +49 (7461) 9 66 37-29

E-Mail info@psm.ms
Web www.psm.ms

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BENEFIT® and **quattro®** are registered trademarks of PSM Medical Solutions, Germany.