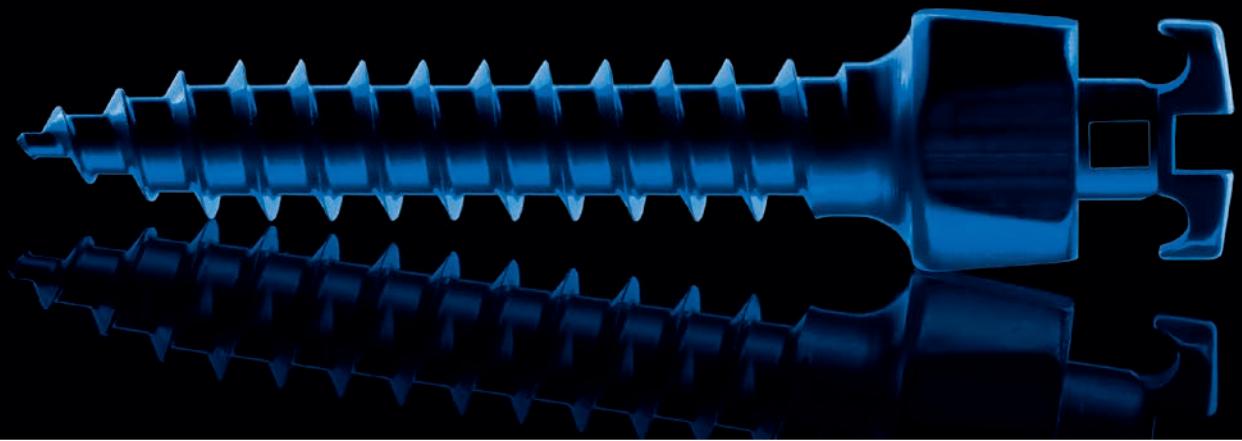
quattro[®].KFO

www.psm.ms – premium implants

made in Germany

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**PSM Medical Solutions – The Precision Screw
Manufacturer**

PSM Medical Solutions, established 1993, was originally specializing in implants and instruments for the bony neurosurgical area. The company based in Tuttlingen, Germany, is part of a medical-economic cluster in and around Tuttlingen, the renowned world center of the medical industry. Many of the surgical instrument manufacturers have transferred their historic handmade craft into the high-tech companies known today.

The Tuttlingen region is known throughout Germany as the epicenter for medical scientific creativity. The local university offers specific courses in the study of the medical device industry. PSM has taken part in this evolution.

In addition to the PSM neurosurgical line, we are now pleased to offer a specialized implant, screw and instrumentation line for orthodontics/oral surgery. Together with our international partners, PSM continues to grow as the trusted partner for clinicians world wide. PSM distinguishes itself from other manufacturers by combining practice, science, technology, material quality and excellent customer service in a cohesive way that truly advances the art of patient care.

**PSM Medical Solutions – The Precision Screw
Manufacturer**

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Orthodontic Mini-Anchor System

The use of Mini-Anchor Screws or Temporary Anchorage Devices (TAD's) has become a standard of care within the orthodontic community over the past few years.

First offered in 2000 the **quattro** TAD design has continuously been changed and improved to become the **quattro** TAD family of today. The **quattro** TAD's even after more than a decade of production are still unmatched in terms of versatility, quality and maximal success rate. With the **quattro mini** screw a new even smaller bracket head design is launched in 2012 offering a new dimension in patient comfort based on the outstanding **quattro** design.

Since year 2000 close to 50.000 **quattro** TAD's have been placed in more than 50 countries worldwide.

The **quattro** System is ideally enhanced with the unique **BENEFit/BENEplate** System (developed in cooperation with University of Düsseldorf, Germany) offering exchangeable abutments and connecting plates. This opens up new possibilities in palatal based appliances popularly used for molar distalization (see QR Video).

quattro and **BENEFit** a complet TAD System using the same basic instrumentation keeping inventory manageable and very cost effective.

Features:

- Clinically Proven Applications
- Titanium Implant Material
- Self-Drilling & Self-Tapping Technique
- Reliable & Secure-Fixed Anchorage
- Innovative Head Attachment Design
- Stronger Conical Body Design
- Deeper Threads & Denser Pitch Design
- Various Screwhead Design
- Available in multiple Sizes

Benefits:

- Provides improved higher strength
- Immediate load-bearing of heavy forces
- Easy to combine with applications of orthodontic accessories
- Better mechanical retention
- Avoids gingival impingement
- Reduced treatment time
- Easy to insert and remove

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quattro plus V

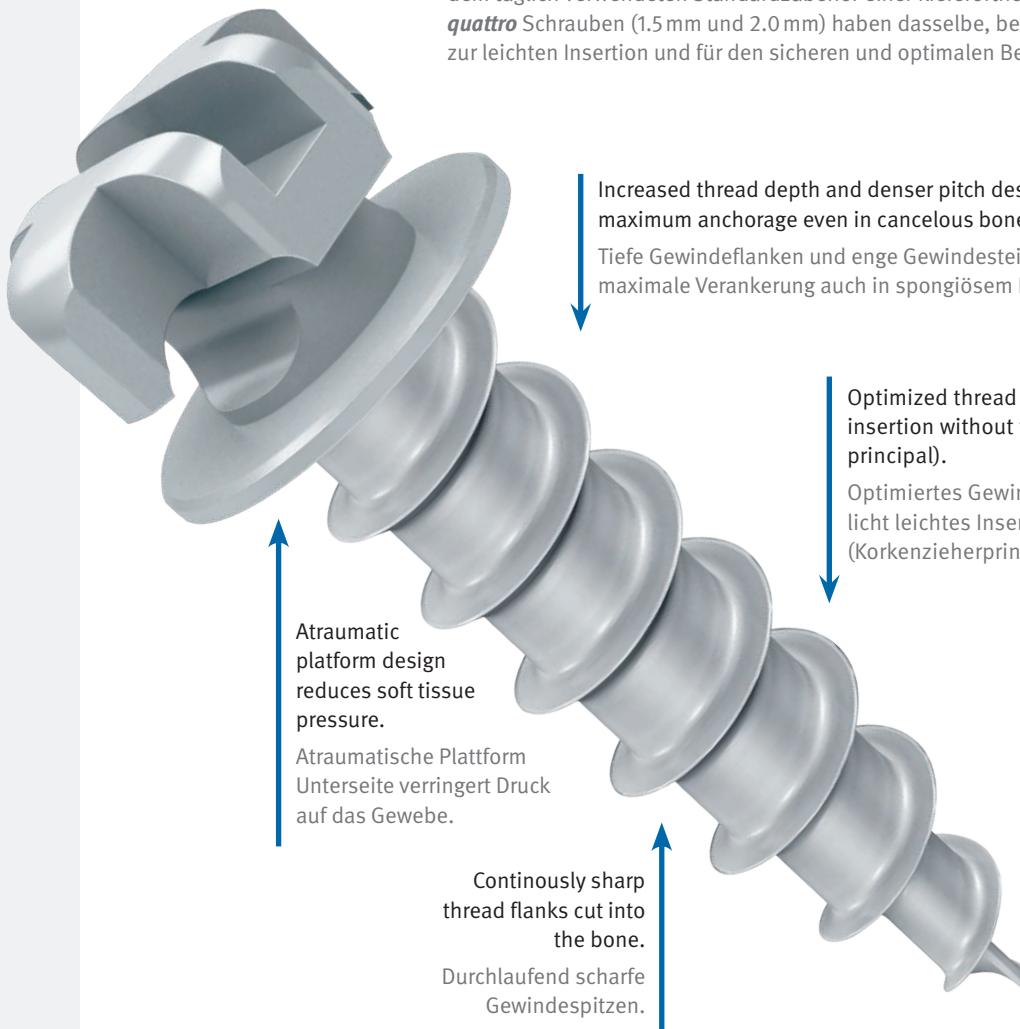
quattro mini

One Screw – Unlimited Possibilities

The **quattro** TAD system offers 5 different screwhead designs which are perfectly adapted to the Orthodontist's appliances used daily. All **quattro** screws (1.5 mm and 2.0 mm) feature the identical and well-proven thread design for easy insertion and most secure and satisfactory treatment results.

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1993	2000	2001	2002	2003	2004	2005
Foundation of PSM Gunningen, Germany.	Introduction of the quattro standard Mini Anchor Screw.		Inventing the quattro hook screw family.			Revolutionizing orthodontic anchorage with quattro plus

Based on the successful **quattro plus** Screw the **quattro mini** Screw has been reduced in size in the head and neck area to meet compatible up with different anatomical conditions.

The design of the selfdrilling thread is identical to the PSM **quattro** and **BENEFIT** Screws which have demonstrated outstanding primary stability and very low failure rates for many years. The extremely sharp threads and the optimized thread pitch facilitate the insertion tremendously and make the **quattro mini** an ideal addition to your selection of Standard Mini Anchor Screws.

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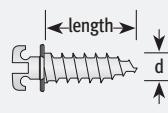
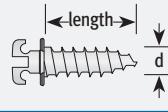
2006

2007/08

Perfecting TAD treatment with
benefit / **beneplates**

2009

quattro plus V

2011

Mentoplaste System

2012

quattro standard

Packaging: 1 ea.
VPE: 1Stk.

Material Titanium
Material Titan

quattro standard

quattro hook

Packaging: 1 ea.
VPE: 1Stk.

quattro hook

Note:

All implants marked with "ST" are supplied in a pre-sterilized single package (see page 11).

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
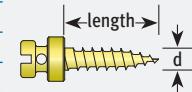
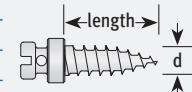
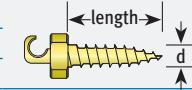
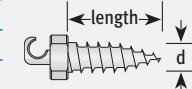
* as long as stock lasts
* Auslaufartikel

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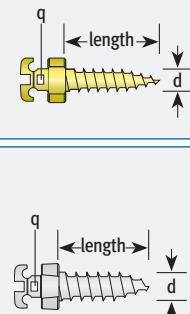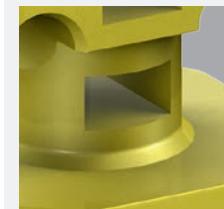
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
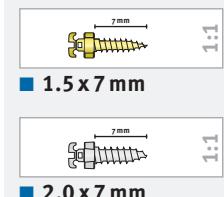
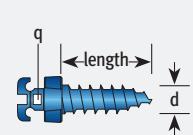
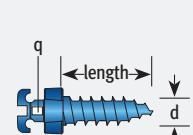
ST

q=.022x.028"

 * as long as stock lasts
 * Auslaufartikel

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■ Note:

All trays are also suitable for use with **BENEFit** and **Mentoplates** implants and instruments.

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73-31960

QB

quattro large sterilization tray for instruments and implants, without content, with lid

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73-31970

QB

quattro small sterilization tray for instruments, without content, with lid

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insert, only

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**INSTRUMENTS
INSTRUMENTE**
■ Note:

All instruments marked with the symbol **QB** can be used with **quattro** and **BENEFit** screws!

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The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use with an angled screwdriver.

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10-63025
QB

Manually turned unit for contra-angled handpieces

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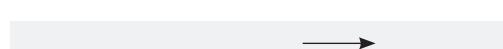
33-54704
QB

 Manually turned unit mod. to Pauls,
 with adjustable torque from 0 – 40 Ncm

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The extension of 16 mm works for all rotary instruments of the **quattro** and **BENEFit** System with a **DENTAL** attachment (max. rpm 600).

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**INSTRUMENTS
INSTRUMENTE**

All **quattro** screwholders are marked with an arrow indicating the slot, hole or hook position on the **quattro** screw during insertion.

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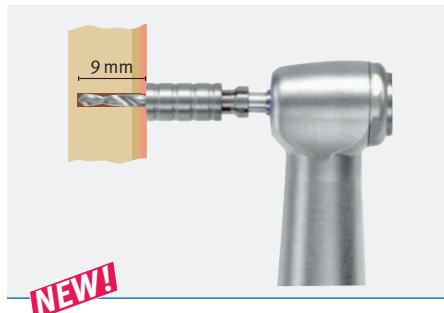
Video

The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate PSM drill*, the working length is adjusted to the desired depth. Removal of the sleeve is easy using the buffer.

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*LOT Nr. >302112 and higher

33-18300

Set, includes 4 sleeves (7 to 13 mm)
without buffer

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10-67513 QB DENTAL

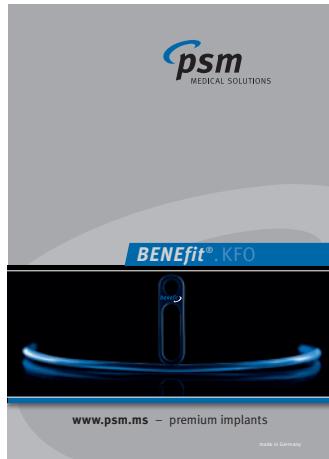
Drill, 1.4x33 mm WL 15 mm,
red, for 2.0 mm screws
Bohrer, 1,4x33 mm AT 15 mm,
rot, für 2,0 mm Schrauben

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COURSES KURSE

psm MEDICAL SOLUTIONS offers courses and webinars in various languages around the world.

Please check the currently available courses on our website <http://www.psm.ms/1/courses/>

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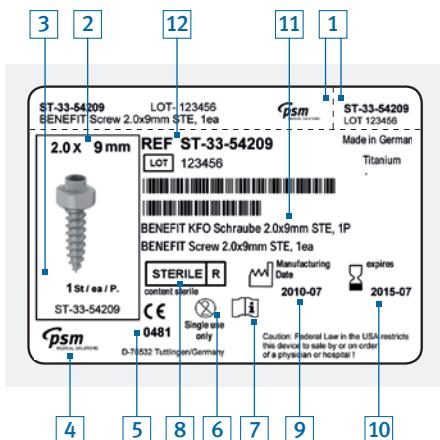
Implants in single sterile packaging

The outer and inner packaging contains labels indicating article number, LOT number, expiration date, etc. for the patient folder and for full traceability.

The sterile screws are contained in a small color-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can easily be removed from the carrier by means of the self retaining screwdriver.

Single sterile packaging

1. 2 self-adhesive labels for patient files
 2. Product size (in mm)
 3. Illustration and packaging
 4. Manufacturer
 5. Reference number of the notified body according to the directive 93/42 EEC
 6. For single use only
 7. Refer to instructions for use
 8. Symbol for gamma sterilization
 9. Date of manufacture
 10. Sterility expiration date
 11. Product description (german, english)
 12. Reference number

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Labels with product illustration for patient files

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© 2016 **psm** MEDICAL SOLUTIONS | Germany

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of **psm** MEDICAL SOLUTIONS. Utilization and copies by third parties have to be authorized by **psm** MEDICAL SOLUTIONS. All rights reserved!

All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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