

# STARLIGHT EXPRESS™

Clear button and clear aligner abutment former with light-polymerisation in pencil-size



This small light-cure pen forms a tooth-colored button quickly and economically.

Simply fill the preformed tip with your light-curing bonding or resin material in syringes and light-cure for a button or a clear aligner abutment that is virtually invisible.

## All advantages at a glance



- Compact pencil size
- Light weight



- Battery powered for convenience
- Fast and easy to use



- Clear button former
- Clear aligner abutment former
- Can be also used on most standard light-curing lamps



- Fast curing time
- Economical

## User Instructions



Simply push the mold on the top of the Starlight Express or on your own light curing lamp.



Fill the preformed mold with your light-curing bonding or resin material in a **syringe**.



Etch the tooth as usual.



Press the Starlight Express™ lightly on the relevant tooth and cure according to the bonding/resin material instructions.



After curing remove the Starlight Express™ carefully with a small tipping motion and reveal your tooth colored button or aligner abutment bonded and ready to use.

## Starlight Express order info

Item #	Article Description	Sales Unit
DP-Starlight Kit	Starlight Express Starter Kit with 2 button formers	1 Kit
DP-Star-Lamp	Starlight Light-Polymerisation Lamp	1 pc
DP-Star-Button	Starlight Express Button Former	2 pc
DP-Star Aligner	Starlight Express Aligner Abutment Fomer	2 pc

